
Panasonic®

Portable Automatic Arm Blood Pressure Monitor
Monitor de Presión Arterial Automático Portátil

Operating Instructions Instrucciones de funcionamiento

Model No. EW3109
Modelo No. EW3109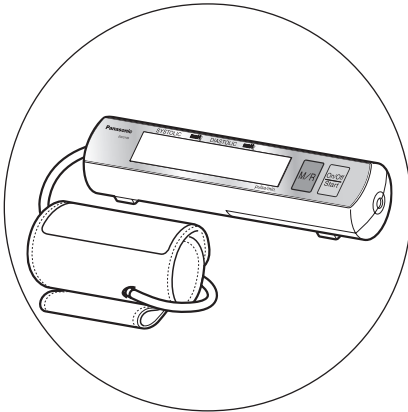

Before operating this device, please read these instructions completely and save this manual for future use.

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Panasonic Oscillometric Portable Automatic Arm Blood Pressure Monitor Model EW3109 is a device intended to measure systolic and diastolic blood pressure and pulse rate of an adult individual by using a pressurized cuff on the arm. The device is not intended for use on infants and children. The device is designed for home use only, not for ambulatory measurement (measurement recorded continuously during the day).

Specification of this device including pulse rate (30 - 160 pulse/min. +/- 5%) are listed in page 19.

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Manual, Electronic or automated sphygmomanometers.

If you suffer from disorder of heart rhythm, known as arrhythmia only use this blood pressure monitor in consultation with your doctor. In certain cases oscillometric measurement method can produce incorrect readings.

Flashing System for hypertensive readings are based on blood pressure values classified in the paper: "JNC 7 Express; The Seventh Report of the Joint National Committee on Prevention, Detection, Evaluation and Treatment of High Blood Pressure; U.S. DEPARTMENT OF HEALTH AND HUMAN SERVICES; National Institute of Health; National Heart, Lung, and Blood Institute; National High Blood Pressure Education Program; NIH Publication No. 03-5233; May 2003." The display values are generally known, but not proven, to be an indicator of your blood pressure.

The EW3109 is not intended to be a diagnostic device. Contact your physician if prehypertensive or hypertensive values are indicated.

Table of Contents

Introduction.....	3
Basics of Blood Pressure	3
Important Instructions Before Use	4
Precautions to Ensure Safe, Reliable Operation	6
Easily Check Your Blood Pressure Readings Against the JNC 7 Classification	8
Parts Identification	9
Installing Batteries	10
When Using the AC Adapter.....	11
Positioning the Pressure Cuff.....	12
For Accurate Measurements	13
Cautions During Measurements	14
Taking a Reading.....	15
Storing Measurement Values	17
Recalling Stored Measurement Values	18
After Use.....	19
Specifications	19
Q & A	20
Care and Maintenance	22
Troubleshooting.....	23

Introduction

Thank you for purchasing the Panasonic Portable Automatic Arm Blood Pressure Monitor EW3109.

Measuring your own blood pressure is an important way of monitoring your health. High blood pressure (hypertension) is a major health problem which can be treated effectively once detected. Measuring your blood pressure between doctor visits on a regular basis in the comfort of your home, and keeping a record of the measurements, will help you monitor any significant changes in your blood pressure. Keeping an accurate record of your blood pressure will help your doctor diagnose and possibly prevent any health problems in the future.

Basics of Blood Pressure

Your heart acts like a pump, sending blood surging through your blood vessels each time it contracts. Blood pressure is the pressure exerted by blood pumped from the heart on the walls of blood vessels. Systolic pressure is the pressure exerted when the heart contracts and pumps blood into the arteries. Diastolic pressure is the pressure exerted when the heart expands, or relaxes. When you or your doctor take your blood pressure, both your systolic and diastolic pressures are measured. If your blood pressure measurement is 120 over 80 (120/80), for example, your systolic pressure is 120 while your diastolic pressure is 80.

Important Instructions Before Use

1. Do not confuse self-monitoring with self-diagnosis. Blood pressure measurements should only be interpreted by a health professional who is familiar with your medical history.
2. If you are taking medication, consult with your physician to determine the most appropriate time to measure your blood pressure. NEVER change a prescribed medication without first consulting with your physician.
3. Blood pressure can vary based on many factors, including age, gender, weight and physical condition. In general, a person's blood pressure is lower during sleep and higher when he or she is active. Blood pressure can change easily in response to physiological changes. The setting in which a person's blood pressure is measured can also affect the results. Having one's blood pressure measured by a healthcare professional in a hospital or clinic can cause nervousness and may result in a temporarily elevated reading. Because blood pressure measurements taken in a clinical setting can vary considerably from those taken at home, a person's blood pressure should be measured not only occasionally in the doctor's office, but also on a regular basis at home. Also, if you find that your blood pressure is lower at home, this is not unusual. To accurately compare with your physician's reading, take your Panasonic blood pressure monitor to your doctor's office and compare readings in this setting.
4. People suffering from cardiac arrhythmia, vascular constriction, liver disorders or diabetes, people with cardiac pacemakers or a weak pulse, and women who are pregnant should consult their physician before measuring their blood pressure themselves. Different values may be obtained due to their condition.
5. Try to take your blood pressure measurements at the same time and under the same conditions every day.
 - The ideal time to measure your blood pressure (to obtain your so-called "base blood pressure") is in the morning just after waking up, before having breakfast and before any major activity or exercise. If this is not possible, however, try to take measurements at a specified time prior to breakfast, and before you have become active. You should relax for about 5 minutes before taking the measurement.
 - The following situations may cause substantial variations in blood pressure readings and should therefore be avoided at least 30 minutes prior to taking your blood pressure.
 - Blood pressure will be higher than usual:

- when you are excited or tense
- when you are taking a bath
- during exercising or soon after exercising
- when it is cold
- within one hour after eating
- after drinking coffee, tea or other beverages containing caffeine
- after smoking tobacco
- when your bladder is full
- when in a moving vehicle

Blood pressure will be lower than usual:

- after taking a bath
 - after drinking alcohol
6. Measurements may be impaired if this unit is used near a television, microwave oven, X-ray equipment or other devices with strong electrical fields. To prevent such interference, use the unit at a sufficient distance from such devices or turn the devices off.
 7. This unit is designed for use by adults. Consult with your physician before using this unit on a child. Do not use on infants or toddlers.
 8. This unit is not suitable for continuous monitoring during medical emergencies or operations.
 9. Do not use the unit for any purpose other than measuring blood pressure. Do not use the unit together with other devices.
 10. Improper handling of batteries may result in battery rupture or in corrosion from battery leakage. Please observe the following to ensure proper use of batteries.
 - a. Be sure to turn off the power after use.
 - b. Do not mix different types or sizes of batteries.
 - c. Change all batteries at the same time. Do not mix old and new batteries.
 - d. Be sure to insert batteries with correct polarity, as instructed.
 - e. Remove batteries when they are worn out, and dispose of them properly according to all applicable environmental regulations.
 - f. Do not disassemble batteries or throw them into a fire.
 - g. Do not short-circuit batteries.
 - h. Do not attempt to recharge the batteries included with the unit.


Precautions to Ensure Safe, Reliable Operation

1. Do not drop the unit. Protect it from sudden jars or shocks.
2. Do not insert foreign objects into any openings.
3. Do not attempt to disassemble the unit.
4. Do not crush the pressure cuff.
5. If the unit has been stored at temperatures below 0°C (32°F), leave it in a warm place for about 15 minutes before using it. Otherwise, the cuff may not inflate properly.
6. Do not store the unit in direct sunlight, high humidity or dust.
7. When cleaning, unplug the AC adapter from the AC outlet before cleaning. Also, do not plug in or unplug the AC adapter with wet hands. Doing so may result in electric shock or fire.
8. Do not spill water on the main body or AC adapter. Doing so may cause electric shock, short-circuiting, or product damage.
9. Do not use the AC adapter if it is damaged or if it plugs into the AC outlet loosely. Doing so may result in electric shock or short-circuiting.
10. If the product does not operate properly or seems strange, stop use and unplug the AC adapter immediately, and request inspection and service. Failure to do so could result in the risk of an accident, electric shock, or fire.
11. When using the AC adapter, make sure it is fully inserted into the AC outlet. Failure to do so could result in electric shock or short-circuiting.
12. When the product is not in use, unplug the AC adapter from the AC outlet. Failure to do so could result in deterioration of the insulation due to dust or moisture, causing a leakage electricity fire.
13. When unplugging the AC adapter, do not pull on the cord. Always grasp the AC adapter body and unplug it. Failure to do so could result in the risk of electric shock, short-circuiting, or fire.
14. Do not insert any object in the AC adapter or allow debris to collect on it. Doing so could result in the risk of electric shock, short-circuiting, or fire.
15. Do not scratch, damage, modify, forcibly bend, pull, or twist the AC adapter cord. In addition, do not place heavy objects on top of the cord or pinch it with objects. Do not wrap the cord around the main body and the AC adapter. Doing any of these things may result in the risk of fire or electric shock.
16. Do not step on the AC adapter. Doing so may cause an accident or injury.
17. Do not use a voltage converter. Doing so may cause fire or electric shock.

18. The AC adapter EW31095 is designed for use only with Panasonic Blood Pressure Monitor, model EW3109. Use with other electrical devices may result in death or serious personal injury.
19. ALWAYS unplug the AC adapter from the power outlet before performing maintenance.
20. Do not let this product to be used by children.
21. Do not let this product to be used by people who cannot express their feelings, or who cannot operate the product by themselves. Such use may result in an accident or injury.
22. Do not allow any material to drape over the AC adapter. Doing so could result in the risk of fire.
23. Do not modify or attempt to repair the product or AC adapter by yourself. Doing so could result in the risk of fire, electric shock, or personal injury.

Easily Check Your Blood Pressure Readings Against the JNC 7[®] Classification

Blood Pressure Categories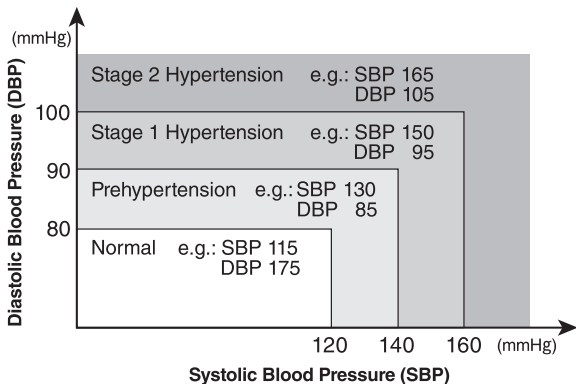

If the two blood pressure measurements (systolic, diastolic) fall into separate categories, your level is classified in the higher of the two categories. For example, a Stage 2 systolic reading, but a diastolic pressure reading in the normal range.


* JNC 7: The Seventh Report of the Joint National Committee on Prevention, Detection, Evaluation and Treatment of High Blood Pressure
National Institute of Health (NIH) Publication; No. 03-5233, May 2003

IMPORTANT:


- Do not be alarmed by temporarily high or low readings because fluctuations in a person's blood pressure are not uncommon. If possible, measure and record your blood pressure at the same time every day, and consult your physician if you have questions or concerns.
- If abnormal variations in blood pressure are observed in measurement, please consult your physician.

Parts Identification

Please check the instrument.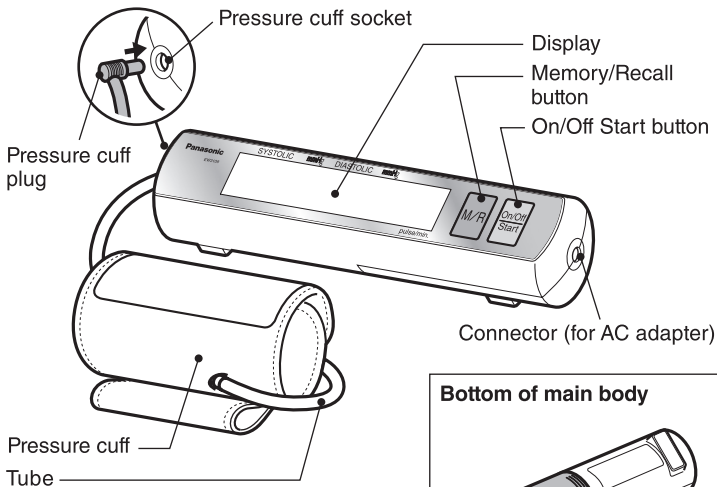

Bottom of main body

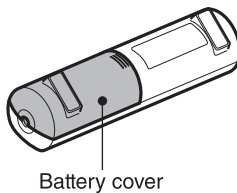
Optional Accessory


Storage pouch

AC adapter EW31095

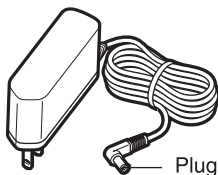
Large Cuff EW3901
35 to 44cm (13-3/4" to 17-1/4")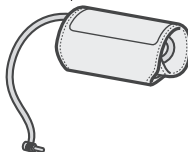

Installing Batteries

* Always use alkaline batteries. (Four AA-size LR6 alkaline batteries)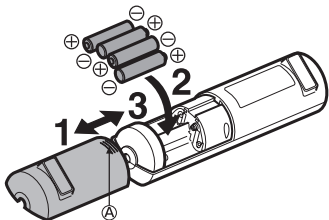


1. Turn the main unit over and remove the battery cover by sliding it in the direction of the arrow while pressing lightly on the (A) mark.
2. Check the polarities of the batteries and install them correctly.
3. Replace the battery cover and slide it closed.

CAUTION:

- Always follow the cautions printed on the batteries.
- Check the polarities of the batteries and install them correctly.
- Remove exhausted batteries promptly.
- If the product will not be used for a long time, remove the batteries.
- Batteries should be used before their expiration date.
Use after the expiration date may result in injury or staining of the surrounding area due to generation of heat, bursting, or leakage.


Replace batteries when:

- The mark shown at left appears during use.
- All displays are lit for a moment after the Start/Stop button is pressed, and then the display goes blank.
- No display appears even if the On/Off Start button is pressed.
- Approximately 500 measurements (approximately 5 months when measurements are taken 3 times per day) can be taken with a set of fresh Panasonic alkaline batteries (four AA-size LR6 batteries). (Measurement conditions: Room temperature 23°C (73°F); Pressurized to 170mmHg; Upper arm circumference: 30cm (11-51/64")) (The batteries included are not guaranteed for the full life of brand new alkaline batteries as specified.)
- When batteries other than Panasonic alkaline batteries are used, battery life may be shorter. When manganese batteries are used, battery life may become as short as 1/6th the life of alkaline batteries.
- Even if the product is not used for a long time, be sure to replace batteries once a year. If battery replacement is delayed, damage due to battery leakage may occur.
- When replacing batteries, always use four fresh alkaline batteries of the same type from the same manufacturer.
- When room temperature is low, battery life may become shorter.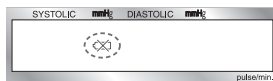

When Using the AC Adapter

* Use only the exclusive AC adapter (EW31095). (See p. 9.)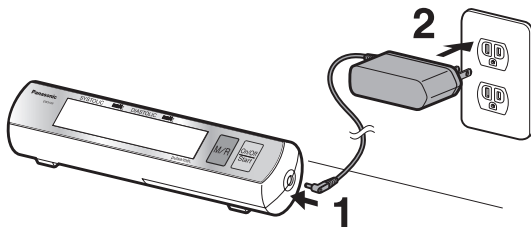


1. Insert the AC adapter plug into the connector on the right side of the main unit.
2. Plug the AC adapter fully into the AC outlet.
(An operating sound may be heard momentarily when the AC adapter is plugged into the outlet.)

Be sure to use only the exclusive AC adapter (EW31095).


Use of other AC adapters could cause an accident.

Positioning the Pressure Cuff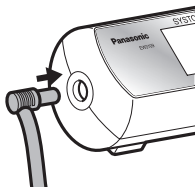


When wrapping around left arm

- Position the tube outlet so that it is on the palm side of the arm in line with the little finger of the hand.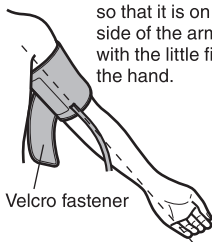

When wrapping around right arm

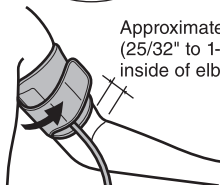
1. Insert the plug into the socket.
 - Insert the pressure cuff plug all the way in.
2. Wrap the cuff around upper arm.
 - Remove your shirt or roll up your sleeve so that the cuff will be on your bare skin.
 - If rolling up your sleeve causes constriction on your arm, remove your shirt for measurements.

The blood pressure monitor can be used by wrapping the cuff on either upper arm. However, since it has been said that the difference in blood pressure between the left and right arms may be around 10mmHg, be sure to always use the same arm for measurements.

3. Fasten so that there is no gap between the cuff and the arm, but be careful that the cuff is not too tight.
 - Be careful not to catch your clothes when wrapping the cuff. (Doing so may lead to lower accuracy.)
 - Press the tape fastener down well.

For Accurate Measurements

Take measurements while sitting quietly and relaxing.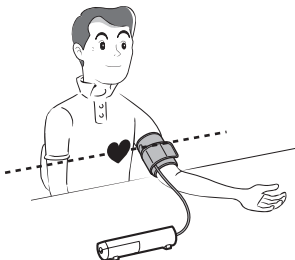

1. Rest your elbows on a table, etc.
2. Position your arm so that the pressure cuff is at the same height as your heart.
3. Position your palm upward, and relax your hand.

Take measurements using the same upper arm in the same position at the same time every day.

- Measurement values will vary according to the height of the pressure cuff.
- Take measurements in the morning immediately after you wake up, when there is no urge to urinate. If this is difficult, take measurements before breakfast and before you have moved a lot.

Relax and sit in a natural posture when taking measurements.

- Always sit quietly for approximately 4 to 5 minutes before taking measurements.
- Relax your body, arm, and fingers and do not move their muscles.
- Do not talk while taking measurements.
- During measurement, do not touch the main body or the tube.


CAUTION:

Keep away from cellular telephones and other sources of electromagnetic radiation. Failure to do so could result in malfunctioning.

Cautions During Measurements

In the following situations, accurate measurements cannot be taken.

Within 1 hour of eating or drinking alcohol

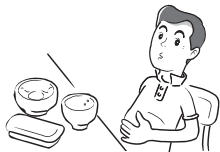Immediately after drinking coffee or tea, or smoking

Immediately after exercising or taking a bath

Wait at least 20 minutes and take a measurement while staying quiet.

In a moving vehicle

In cold places

Take measurements at ambient temperatures of 10°C to 40°C (50°F to 104°F).

When you need to urinate


Wait several minutes after urinating before taking measurements.

Taking a Reading

Measurement

1. Press the On/Off Start button.
 - The pressure cuff will automatically be pressurized and a measurement will be taken.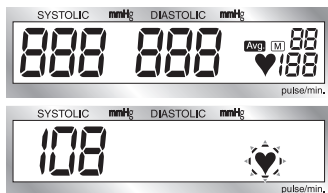

The entire display will light for approximately 1 second.

- If there is air left in the pressure cuff, the fully lit time will be longer.

The heart mark ♥ will blink when pulse is detected during pressurization.

In the process of pressurization, changes in pressurization speed and operating sounds are normal and necessary for the purpose of measurement during pressurization.

2. When measurement has been completed, the blood pressure and pulse values will be displayed.


- Pressure will be automatically released from the pressure cuff.
 - If the measured value is in the high blood pressure region, the values will blink (for approximately 6 seconds). (For how to read the blood pressure display, see p. 16.)
 - The **[M]** mark will blink. (For storing measurement values, see p. 17.)
 - The pulse rate value is the beats per minute calculated based on the pulse rate during measurement.
3. To switch off the blood pressure monitor, press the On/Off Start button.
 - If you forget to switch it off, the blood pressure monitor will switch off automatically after approximately 5 minutes.

Flash warning system for hypertensive readings

If the measured values fall within the hypertension range, the reading displayed on screen flashes on and off to alert user.

According to JNC 7 Classification, values consistently in excess of 140 mmHg (Sys.) and/or 90 mmHg (Dia.) are considered to constitute high blood pressure.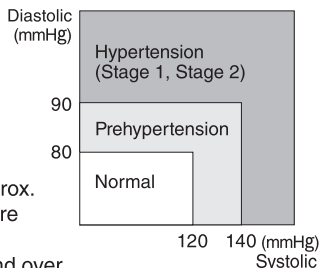

- Blood pressure readings will flash for approx. 6-seconds when in the high blood pressure range.

Systolic blood pressure: 140 mmHg and over

Diastolic blood pressure: 90 mmHg and over

(Only when measurement is complete)

When an Error Occurs During Measurement

An E will appear on the display to indicate that measurement was unsuccessful and should be performed again.

Before performing another measurement, always press the On/Off and Start button to turn off the unit and make sure to allow a rest period of 4 to 5 minutes before starting again.

Please consult your doctor without delay if the blood pressure Indicator repeatedly draws attention to elevated blood pressure values.

Advice:

In general, blood pressure measurements are lower for the second measurement because you are less nervous and are more tranquil.

For people who easily get tense, measuring twice each time is very effective. When two measurements are taken, normally the smaller measurement is stored.

When taking two measurements, always stay quiet for 4 to 5 minutes after the first measurement before taking the second measurement.

Storing Measurement Values

This product can store 90 measurement values.

After measurement, press the Memory/Recall button while the (M) mark is blinking.

Storing is completed.

- Up to 90 measurement values can be stored. When the memory is full and additional values are stored, the oldest value in memory will be deleted and the new value will be stored.
- If measurements could not be taken properly (the E mark is shown instead of the blood pressure value), measurement values cannot be stored.
- * However, even if the E mark is shown for the blood pressure value, the pulse value will still be stored.

Number of stored data

To erase all measured values stored in memory:

1. Press the Memory/Recall button. The stored measurement values will be displayed.
2. Press the same Memory/Recall button again, and hold it pressed until the measurement values disappear (approximately 3 seconds or more).
 - * It is not possible to erase single measurement values.
3. The (M) mark and 00 will be shown when all data is deleted.

To switch off the blood pressure monitor, press the On/Off Start button.

- If you forget to switch it off, the blood pressure monitor will switch off automatically after approximately 5 minutes.

Recalling Stored Measurement Values

Measurements can be recalled after measurement is completed or after the power has been switched off.

- If no measurement values are stored, no value will be shown.

1. Press the Memory/Recall button for the person whose data you want to recall.

The average of all measured values in memory will be shown.

- **Avg.** will be shown at the top right of the display.
- For example, if 90 measured values are stored, the average of the 90 measurements will be shown. (If only 1 measurement is stored, the **Avg.** will not be shown.)

2. Press the Memory/Recall button again.

Each time you press the Memory/Recall button, the previously stored measurement values will be shown in order, starting with the newest data.


Do not press the Memory/Recall button for an extended length of time. If you hold the button pressed for approximately 3 seconds or longer, all data in memory will be erased, so please be careful.

3. To cancel measurement value recall, press the On/Off Start button.

- If you forget to switch it off, the blood pressure monitor will switch off automatically after approximately 30 seconds.

After Use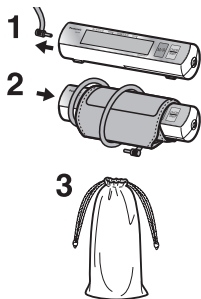

Storage

1. Unplug the plug from the main body.
2. Roll up the pressure cuff loosely and place the main unit inside it.
3. Place the cuff and main body in the storage pouch.
(Please take care not to create kinks in the tube.)
 - Do not place the AC adapter in the storage pouch.

Specifications

Power source	DC 6V (Four AA/LR6 size alkaline batteries) AC 120V 60Hz (when using optional AC adapter)
Display	Digital LCD
Method of measurement	Oscillometric system
Measurement range	Blood pressure: 0 to 280mmHg Pulse rate: 30 to 160 beats per minute
Accuracy	Blood pressure: Within ± 3 mmHg Pulse rate: Within $\pm 5\%$
Operation temperature/ humidity range	10 to 40°C (50 to 104°F), 15 to 90% RH
Storage temperature/ humidity range	-20 to 60°C (-4 to 140°F), 10 to 95% RH
Measurable arm circumference	Approx. 20 to 40cm (7-3/4" to 15-3/4")
Weight	Approx. 260g (9.2 oz.) (not including batteries)
Dimensions	5.2 x 20.8 x 5.2cm (2" x 8-2/8" x 2")
Protection against electric shock	Internally powered equipment, Type BF applied part

Q & A


Q Why are the blood pressure values measured at home different from those measured at the doctor's office?

A Blood pressure varies 24 hours a day.

In addition, it can change greatly depending on the weather, your mental state, exercise, etc.

In particular, there is the phenomenon of “White coat hypertension” at doctor's offices, where the blood pressure measured in a doctor's office is higher because of stress or worry.

In addition, if the blood pressure measured at home is sometimes higher and sometimes lower than the blood pressure measured at the doctor's office, the following can be considered. Please pay attention to these points when taking measurements.

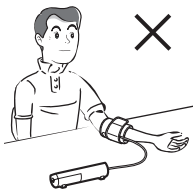Is the pressure cuff wrapped firmly?

- Make sure that the pressure cuff is not wrapped too loosely or too tightly.
- Is the pressure cuff wrapped around the upper arm? Make sure it is not partly covering the elbow. For how to wrap the pressure cuff correctly, refer to p. 12.

Were you worried or upset during measurements?

Before taking measurements, take 2 or 3 deep breaths and relax to stabilize blood pressure. In addition, it is recommended that you sit quietly for about 4 to 5 minutes before taking measurements.

Q When should I measure my blood pressure?

A We recommend three measurements a day.

First Upon Waking (Because, with almost no influence from body movement, it's the blood pressure is closest to that at night.)


Second... Before Lunch (Because it provides blood pressure values close to the highest of the day.)

Third At Bedtime (So as to obtain data close to your average blood pressure values during the day.)

This way, by taking measurements morning, noon and night, you can use the unit to learn the patterns of and monitor your blood pressure.

Q Why are measured blood pressure values different each time I take a measurement?

A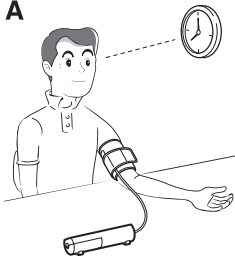


Blood pressure changes even within a single day, and it is different depending on the measurement posture. Because of this, when taking measurements, be sure to always take measurements under the same conditions.

For persons taking blood pressure medication, blood pressure may change greatly due to the effects of the medication.

When taking 2 measurements in succession, be sure to stay quiet for 4 to 5 minutes after the first measurement before taking the next measurement.

Q Why are the measured values different for the left and right arm?

A

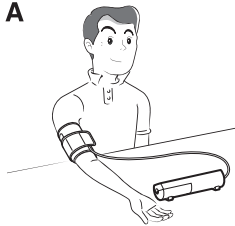It is said that differences in blood pressure values between the left and right arms may be around 10mmHg. The blood pressure can be measured in either the left or right arm, but all measurements should be taken using the same arm for comparative consistency.

Care and Maintenance

Do not attempt to disassemble, repair or modify the unit.

- Doing so may cause fire or cause the unit to malfunction. It may also lead to injury.

Do not fold the pressure cuff back against itself.

Always use the pressure cuff on your arm and nowhere else.

- Either of these actions may cause the device to malfunction.

Do not apply excessive force to the unit or drop the unit.

- Doing so may cause damage.

Do not insert dust or foreign objects into the unit.

- Doing so may result in damage.

If the unit is stored at temperatures below the freezing point, do not use it immediately.

Leave it in a warm place for at least an hour before use.

- If the unit is not allowed to warm up, it may not pressurize.

Do not touch the unit when measurement is in progress.

- Doing so may cause incorrect readings.

If the unit becomes dirty, clean it with a soft cloth moistened with warm water or soapy water.

(Do not use alcohol, benzene, or paint thinner.)

- Use of such chemicals may result in cracking or discoloration.

When the unit will not be used for a long time (30 days or more), be sure to remove the batteries.

- Otherwise, the batteries may leak and damage the unit.

The pressure cuff cannot be washed.

For storage, avoid high temperatures, high humidity, and direct sunlight.

- Storage under such conditions may result in damage.

Troubleshooting

Display	Status before error	Cause and solution	If problem continues.
<p>Remeasure mark E is displayed.</p> 	<p>▶ Pressurization to more than 280mmHg was performed.</p> <p>▶ Pressure decreased suddenly and the remeasure mark E appeared</p> <p>▶ Measurement was taken to 20mmHg and then the remeasure mark E appeared.</p> <p>▶ The heart mark ♥ blinked only a few times or not at all.</p> <p>▶ Pressurization doesn't occur even after waiting several dozen seconds.</p>	<p>▶ Were measurements taken according to the proper procedure and in the correct posture? See p. 12 and 13.</p> <p>▶ Was the pressurization cuff wrapped properly? See p. 12.</p> <p>▶ Tube may not be connected sufficiently. Check the tube connection. See p. 12.</p>	
Symptom	Possible cause	If problem continues.	Ask your doctor for advice.
<p>Systolic or diastolic blood pressure value is high or low.</p> <p>Blood pressure value is abnormally high or low.</p> <p>Value is different from that measured by the doctor.</p> <p>Measurement values are different with each measurement.</p> <p>No display appears even when the On/Off Start button is pressed.</p> <p>When the On/Off Start button was pressed, all displays became lit and then the display disappeared.</p> <p>Instrument is broken.</p>	<p>• Position of pressure cuff is low.</p> <p>• Pressure cuff is not wrapped correctly.</p> <p>• Position of pressure cuff is high.</p> <p>• You moved or talked while measurements were being taken. See p. 12 and 13.</p> <p>Posture is different for each measurement. See p. 13.</p> <p>• Blood pressure varies in response to minute changes in your mental state, such as your reaction to a doctor's presence.</p> <p>• Sit quietly for 4 to 5 minutes and then take another measurement. See p. 13.</p> <p>• Batteries are exhausted.</p> <p>• Batteries are installed with the polarities wrong. See p. 10.</p>		
<p>Take it to the store where it was purchased for testing and repairs.</p>			

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Panasonic Corporation of North America
Panasonic Latin America, S.A.